English

BARD® Urological Catheter

Warning: Do not use ointments or lubricants having a petrolatum base. They will damage latex. Visually inspect the product for any imperfections or surface deterioration prior to use.

- Use luer tip syringe to inflate with stated mL of sterile water.
 - OR
 - For pre-filled products, remove clip and squeeze reservoir to inflate with stated mL of sterile water.

For urological use only. To deflate catheter balloon: Gently insert a luer slip tip syringe in the catheter valve. Never use more force than is required to make the syringe "stuck" in the valve. Allow the pressure within the balloon to force the plunger back and fill the syringe with water. *If you notice slow or no deflation, re-seat the syringe gently.* Use only gentle aspiration to encourage deflation if needed. Vigorous aspiration may collapse the inflation lumen, preventing balloon deflation. If necessary, contact adequately trained professional for assistance, as directed by hospital protocol.

Should balloon rupture occur, care should be taken to assure that all balloon fragments have been removed from the patient.

Female Use Only

Syringe of Sterile Water for balloon inflation

Refer to Direct Unit Label for Product Content and Gender Specific Use where applicable.

LATEX Contains or Presence of Natural Rubber Latex. **Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.**

 Single use only. Do not reuse.  Do not resterilize.

This is a single use device. Do not resterilize any portion of this device. Reuse and/or repackaging may create a risk of patient or user infection, compromise the structural integrity and/or essential material and design characteristics of the device, which may lead to device failure, and/or lead to injury, illness or death of the patient.

LOT Lot Number	REF Reorder Number	 Use by Date
 Caution, consult accompanying documents.	 Phthalate Free	 Consult instructions for use.
 Storage: Store catheters at room temperature away from direct exposure to light, preferably in the original box.	 Keep away from sunlight	 Do Not Use if Package is Damaged.

STERILE R Sterilized by radiation **OR** **STERILE EO** Sterilized by Ethylene Oxide

Refer to Direct Unit Label for Sterilization Method Utilized

 BARD, BARDEX, BARDIA, BIOCATH and LUBRICATH are trademarks and/or registered trademarks of C. R. Bard, Inc.

Warning: After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable laws and regulations.

BARD
Manufacturer: C. R. Bard, Inc.
EC MPF
Forest Limited
Bard
UK
Forest House
Crawley, West Sussex UK RH11 9BP
1 800 526 4455
+44 1293 527 888
www.bardmedical.com
CE
0086

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